


## One set contains:




Assemble in a clean work environment. Wear gloves and a face mask.  
Video instructions are available at <https://bit.ly/shield-video>.




1 Peel off the protective film from both sides of the shield.

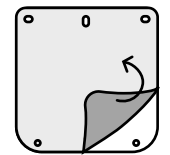


2 Insert the nuts into the nut casings.

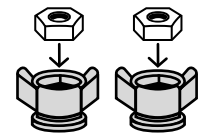

3 Rotate the shield perpendicular to the headband, attach it to the pin in the frontal part and fix it in its final position by rotating it back 90 degrees.

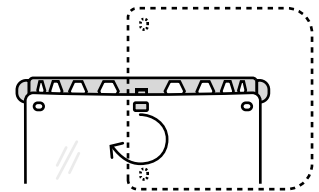


4 Insert the screw from the inside through the hole at the back of the headband and mount the shield on it. Secure the shield to the headband by screwing the nut housing with the nut inserted.

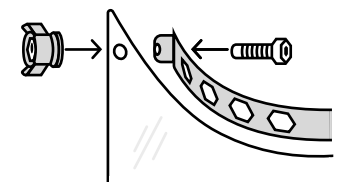

Repeat the step on the other side of the headband.



5 Attach the elastic band to the nut casings on both sides of the headband.

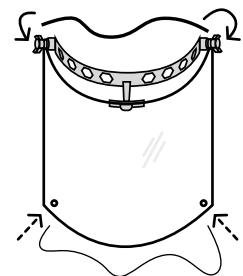

In case you need to adjust the shape of the shield, pull a piece of thread through the holes in lower corners and tie it up. The shield will be curved depending on the degree of tightening.

## Maintenance:

Use common disinfectants such as ethanol for disinfecting the face shield.

The shield can be sterilized in systems using low-temperature hydrogen peroxide plasma (STERRAD).

Do not expose the shield to temperatures exceeding 60 °C / 140 °F.